

SCANNER TIP CLEANING INSTRUCTIONS

DO NOT PLACE IN ULTRASONIC CLEANER and DO NOT AUTOCLAVE

NOTE: Instructions were validated using Maxicide Plus w/ Activator (3.4% Gluteraldehyde) disinfectant (Henry Schein Dental #102-2865).

1. Clean the tip for 2 minutes under running tap water at 22 - 25°C (4 liters/minute) to remove debris.
2. Test the potency of your activated MaxiCide Plus prior to disinfection. (ie. MetriTest Strips by Metrex (HSD #602-3437)
3. Immerse the tip in activated Maxicide Plus for 120 minutes (2 hours). **NOTE:** Do not leave the tips soaking overnight.
4. Remove tip from the disinfectant.
5. Rinse disinfected part in three separate copious amounts of distilled water. (1000 ml volume of water; minimum of 2" head height while part is immersed).
6. Agitate the tip in the water for 30 seconds and then let it stand in the water for another 30 seconds.
7. Repeat the agitation and soak in each of the other two containers with fresh distilled water.

STORAGE

1. Wipe water off mirror using optical wipes.
2. Optional: Insert tip into a sterilization pouch.
3. Store for later use.

PRIOR TO USE

1. Ensure the tip is dry before connecting it to the base.
2. Visually confirm the scanning tip mirror is clean and dry.
 - To clean the mirror, use a Kimwipe, 2x2 gauze, or optical cloth with a little alcohol to gently clean the mirror.
 - Dry the mirror with a dry Kimwipe, 2x2 gauze, or dry optical cloth.